

PLEASE NOTE,

Amendment to the Instruction for Use for LUCAS with serial numbers less than 1406 4000 regarding ventilation during LUCAS compressions.

Ventilation with LUCAS in a non intubated patient

When LUCAS is used on a non intubated patient the device can safely be stopped allowing intermittent manual ventilation. Switch off the device by turning the ON/OFF knob to the **Lock** position, ventilate and turn the ON/OFF knob back to the **Active** position again.

Ventilation with LUCAS in an intubated patient (advanced airway)

When using LUCAS on an intubated patient, there is no need to interrupt LUCAS chest compressions for ventilation. Ventilate according to guidelines for advanced life support.